COSMOS CREATION GLAZE

PORTUGUÊS / ENGLISH / ESPAÑOL

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1. PRODUCT CLASSIFICATION

1.1. Technical name

Photopolymerizable pigments for 3D printing.

1.2. Product trade name

COSMOS CREATION GLAZE - Shade for make-up.

1.3. Description of the physical principle and fundamentals of the product's technology, applied to its operation and action
COSMOS CREATION GLAZE is a photopolymerizable resin for all types of work on acrylics, composite resins and pieces produced by 3D printing.

1.4. Technical specifications and characteristics

1.4.1. Composition

Components
Oligomers
Monomers
Photoinitiators
Stabilizer

2. STORAGE

Store in a dry place at a temperature between 35.6°F and 77°F.

3. INSTRUCTIONS FOR USE

- Apply a thin layer of COSMOS CREATION GLAZE to a work surface and leave to act for 30 seconds before polymerizing.
- Photopolymerize in a UV light chamber with at least 36W of power for 2 minutes.
- If necessary, apply a new layer and photopolymerize again for 2 minutes.

4. PRECAUTIONS, RESTRICTIONS AND WARNINGS

4.1. Precautions

- The product should only be used by professionals.
- Keep the product out of the reach of children.
- The product must not be used after its expiry date.
- Do not use the product if its packaging is damaged.

4.2. Restrictions and warnings

Do not use on people with a history of allergy to acrylates and methacrylates.
Contraindication: Not indicated for intraoral application on composite resin veneers for clinical use.

5. INDICATION

Characterization of dental provisional.
Characterization of fixed and removable prostheses.

6. VALIDITY

For manufacturing and expiry dates, see packaging.

7. PRODUCT REGISTRATION

Reg. Anvisa No: 81009350022.

Yller Biomaterials S/A is not responsible for damages, of any nature, caused by incorrect use or use not provided for in the Instructions for Use.

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