COSMOS PLUS CREATION

PORTUGUÊS / ENGLISH / ESPAÑOL

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1.PRODUCT CLASSIFICATION

1.1. Technical name

Photopolymerizable pigments for 3D printing.

1.2. Product trade name

- Cosmos Plus Creation Gingiva Paste G1 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Paste G2 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Paste G3 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Paste G4 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Flow G1 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Flow G2 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Flow G3 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Flow G4 (1 syringe with 2.5g).
- Cosmos Plus Creation Gingiva Paste Kit (1 syringe 2.5g Cosmos Plus Creation Gingiva Paste G1 + 1 syringe Cosmos Plus Creation Gingiva Paste G2 + 1 syringe Cosmos Plus Creation Gingiva Paste G3 + 1 syringe Cosmos Plus Creation Gingiva Paste G4).
- Cosmos Plus Creation Gingiva Flow Kit (1 Cosmos Plus Creation Gingiva Flow G1 Syringe + 1 Cosmos Plus Creation Gingiva Flow G2 Syringe + 1 Cosmos Plus Creation Gingiva Flow G3 Syringe + 1 Cosmos Plus Creation Gingiva Flow G4 Syringe).
- Cosmos Plus Creation Gingiva Kit (1 Cosmos Plus Creation Gingiva Flow G1 Syringe + 1 Cosmos Plus Creation Gingiva Flow G2 Syringe + 1 Cosmos Plus Creation Gingiva Flow G3 Syringe + 1 Cosmos Plus Creation Gingiva Flow G4 Syringe + 1 Cosmos Plus Creation Gingiva

Paste G1 Syringe + 1 Cosmos Plus Creation Gingiva Paste G2 Syringe + 1 Cosmos Plus Creation G Gingiva Paste G3 + 1 Syringe Cosmos Plus Creation Gingiva Paste G4 + 1 Bottle Cosmos Plus Creation Glaze 5ml).

- Cosmos Plus Creation Black (1 syringe with 2.5g).
- Cosmos Plus Creation Brown (1 syringe with 2.5g).
- Cosmos Plus Creation White (1 syringe with 2.5g).
- Cosmos Plus Creation Red (1 syringe with 2.5g).
- Cosmos Plus Creation Blue (1 syringe with 2.5g).
- Cosmos Plus Creation Yellow (1 syringe with 2.5g).
- Cosmos Plus Creation A (1 syringe with 2.5g).
- Cosmos Plus Creation B (1 syringe with 2.5g).
- Cosmos Plus Creation C (1 syringe with 2.5g).
- Cosmos Plus Creation D (1 syringe with 2.5g).
- Cosmos Plus Creation 10 Colours Kit (1 syringe Cosmos Plus Creation Black 2.5g + 1 syringe Cosmos Plus Creation Brown 2.5g + 1 syringe Cosmos Plus Creation White 2.5g + 1 syringe Cosmos Plus Creation Red 2.5g + 1 syringe Cosmos Plus Creation Blue 2.5g + 1 syringe Cosmos Plus Creation Yellow 2.5g + 1 syringe Cosmos Plus Creation A 2.5g + 1 syringe Cosmos Plus Creation B 2.5g + 1 syringe Cosmos Plus Creation C 2.5g + 1 syringe Cosmos Plus Creation D 2.5g + 1 bottle Cosmos Plus Creation Glaze 5ml).
- Cosmos Plus Creation Kit (1 syringe 2.5g Cosmos Plus Creation Gingiva Paste G1 + 1 Syringe Cosmos Plus Creation Gingiva Paste G2 + 1 Syringe Cosmos Plus Creation Gingiva Paste G3 + 1 Syringe Cosmos Plus Creation Gingiva Paste G4 + 1 Syringe Cosmos Plus Creation Gingiva Flow G1 + 1 Syringe Cosmos Plus Creation Gingiva Flow G2 + 1 Syringe Cosmos Plus Creation Gingiva Flow G3 + 1 Cosmos Plus Creation Syringe Gingiva Flow G4 +

1 Cosmos Plus Creation Syringe Black 2.5g + 1 Cosmos Plus Creation Syringe Brown 2.5g + 1 Cosmos Plus Creation Syringe White 2.5g + 1 Cosmos Plus Creation Syringe Red 2.5g + 1 Cosmos Plus Creation Syringe Blue 2.5g + 1 Cosmos Plus Creation Syringe Yellow 2.5g + 1 Cosmos Plus Creation A syringe 2.5g + 1 Cosmos Plus Creation B syringe 2.5g + 1 Cosmos Plus Creation C syringe 2.5g + 1 Cosmos Plus Creation D syringe 2.5g + 1 Cosmos Plus Creation Glaze bottle 5ml).

1.3. Description of the physical principle and fundamentals of the product's technology, applied to its operation and action

Cosmos Plus Creation are light-curing pigments for characterization of dental elements manufactured using 3D printing. They allow you to customize, create effects and change shades in dental prostheses. Available in flow, paste (high viscosity) and shade (for chroma adjustments) versions.

1.4. Technical specifications and characteristics

1.4.1. Composition

Components
Monomers
Inorganic Fillers
Photoinitiators
Pigment

2. STORAGE

Store in a dry place at a temperature between 35.6°F and 86°F. Do not expose to sunlight.

3. INSTRUCTIONS FOR USE

Note: Before beginning the staining process, make sure that all steps related to preparing the restoration have been completed.

- To begin the staining process, select the desired Cosmos Plus Creation shade and the post-cure you will be using:

- Post-curing Anycubic (Wash & Cure 3.0 Model):

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Use all LEDs turned on + Flexicure.
- Cure for 1 minute between layers.
- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.
- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an additional 30 minutes in the post-curing chamber.
- Afterward, remove the excess glycerin and submerge in 70% alcohol for 10 minutes. Allow to dry at room temperature.

- Post-curing Pionext (Model UV 01):

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Use all LEDs turned on at 73.4°F.
- Cure for 1 minute between layers.
- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.
- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an additional 5 minutes in the post-cure chamber.
- Afterward, remove the excess glycerin and submerge in 70% alcohol for 10 minutes. Allow to dry at room temperature.

- Post-curing Espectra (Model UV 01):

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Use all LEDs turned on at 73.4°F.
- Cure for 1 minute between layers.
- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.
- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an

additional 5 minutes in the post-cure chamber.

- Afterward, remove the excess glycerin and submerge in 70% alcohol for 10 minutes. Allow to dry at room temperature.

- Post-curing Pcure:

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Use the "Upper" + "Lower" mode at 100%.
- Cure for 1 minute between layers.
- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.
- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an additional 5 minutes in the post-curing chamber.
- Afterward, remove the excess glycerin and submerge in 70% alcohol for 10 minutes. Allow to dry at room temperature.

- Elegoo Post-Curing (Wash & Cure Mercury Plus V3.0 Model):

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Cure for 1 minute between layers.
- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.
- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an additional 30 minutes in the post-curing chamber.
- Afterward, remove excess glycerin and soak in 70% alcohol for 10 minutes. Allow to dry at room temperature.

- Post-Curing in a 72W or 36W UV Camera:

- Using a brush, apply a thin layer of Cosmos Plus Creation; each layer should not exceed 0.3mm in thickness.
- Cure for 1 minute between layers.

- Once the desired result is achieved, apply a thin layer of Cosmos Plus Creation Glaze.

- Cure for 1 minute between layers.
- Next, completely submerge the piece in glycerin and cure for an additional 30 minutes in the post-cure chamber.
- Afterward, remove excess glycerin and soak in 70% alcohol for 10 minutes. Allow to dry at room temperature.

4. PRECAUTIONS, RESTRICTIONS AND WARNINGS

4.1. Precautions

- When handling this product, it is recommended to use personal protective equipment, such as protective gloves and safety goggles.
- Wash hands and other exposed areas thoroughly after handling the product.
- Keep the product out of the reach of children.
- The product must not be used after its expiry date.
- Do not use the product if its packaging is damaged.

4.2. Restrictions and warnings

- For professional use only.
- This product is contraindicated for people with suspected allergies or hypersensitivity to the chemical components of the material.

5. INDICATION

- Characterization of dental provisionals.
- Characterization of fixed and removable prostheses.
- Characterization of prostheses and crowns.

6. VALIDITY

For manufacturing and expiry dates, see packaging.

7. PRODUCT REGISTRATION Reg. Anvisa No: 81009359011.

Yller Biomateriais S/A is not responsible for damages, of any nature, caused by incorrect use or use not provided for in the Instructions for Use.

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